

## FAS SEAT BELT BUCKLE GUARD

Model No. **SVBG-U101** 58 mm W x 36 mm H x 60 mm D

Patent Number: (Australia) AS2005100698  
(China) 200530063856 & 200520064194.8  
(USA) 11/470999


This Australian designed product offers protection to those at risk of undoing the seat belt buckle accidentally in the car. The see-through design complies with Australian/New Zealand Standard AS/NZ 4370:1996 - Restraint of Children with Disabilities in Motor Vehicles. It is recommended by the Australian/New Zealand Standard AS/NZ 4370:1996 that a **medical certificate** from medical practitioner and a proper prescription form to be completed before we can supply this Buckle Guard. A copy of the medical certificate should be kept in the vehicle that the user is in. Periodic review of use is also recommended. A set of prescription form and information sheets are available from our office or can be down loaded from our web site [www.fasequipment.com](http://www.fasequipment.com).

Once this Buckle Guard is applied, the user will not be able to undo the seat belt buckle. The seat belt buckle can be safely released by inserting any key-size object in one of the slots. A snap-off wedge spacer is provided for taking up loose space and allowing the buckle to rest snugly inside the Buckle Guard. The universal size will suit most car seat belts. For those with a non-standard sized seat belt please contact us for further information.

The Buckle Guard is made of ABS plastic hence is very difficult to break. The package now comes with a webbing cutter for emergency use, a wedge spacer, and a warning label.




Buckle Guard protects the child from accidentally or intentionally undoing the seat belt buckle in the car. See-through design ensures it is functioning.

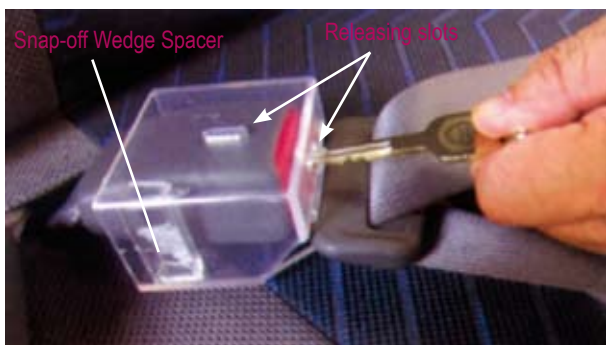

The Buckle Guard comes with a snap-off wedge spacer for taking up spaces between the buckle and Buckle Guard. Two releasing slots are available on the Buckle Guard.

## 4 EASY STEPS TO INSTALL THE BUCKLE GUARD

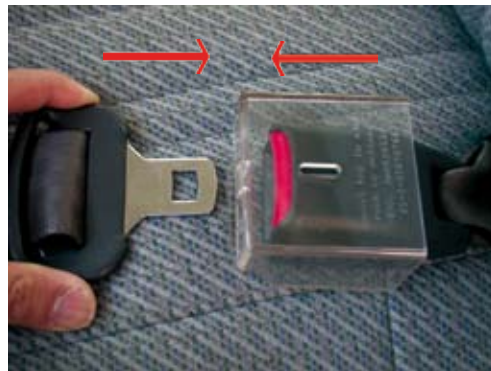


1. Slide buckle into Buckle Guard, fasten buckle by inserting the tongue through the half opening end.



2. Insert Wedge-shape Spacer (can be broken into two pieces), slide along wall to take up loose space.



3. Fix spacer with adhesive tape.

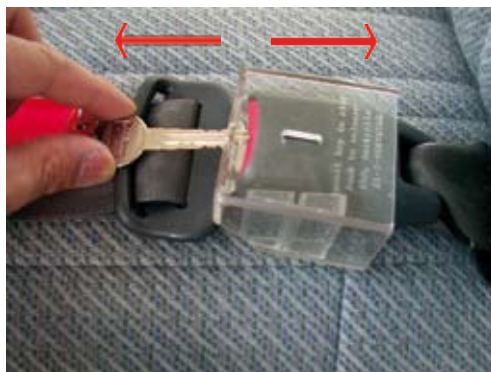

4. Insert a key type object into one of the two slots to release the seat belt buckle .